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Morva Prep

15% EDTA, 10% Urea Hydrogen Peroxide

EDTA Cream for Root Canals Cleaning, widening and Preparation

For Chemo-Mechanical Preparation of Root Canals

Ingredients:

15% EDTA, 10% Urea Hydrogen Peroxide, Cream foundation, Lubricant

Morva Prep contains Ethylenediamine Tetraacetic Acid and Urea hydrogen Peroxide. The chelating ability of this product will help remove the calcium salts from pulp stones and from the walls of the root canal. This action aids in the cleansing and shaping of the canal. Morva Prep also contains urea-peroxide in a special water soluble vehicle that lubricates the canal permitting instruments to more easily enter even fine canals and reduce the incidence of breakage.

Morva Prep used with sodium hypochlorite solution gives you these advantages:

- ▶ Facilitates cleansing and shaping of the Root Canal. The EDTA helps remove calcium salts from the calcifications and from the canal surface, permitting the reamers and files to more rapidly cleanse and shape the canal.
- ▶ Pulp Tissue)Vital/ necrotic tissue(is more easily removed from the canal when irrigated with sodium hypochlorite solution, the oxygen bubbles released from the urea peroxide float out the pulp tissue, shavings and other debris.
- ▶ Enhances the penetration of medicaments throughout the root canal system by its ability to more completely clean the canal and open the dentinal tubules.
- ▶ The product has a dense, creamy consistency but after placing it in the canal it becomes like liquid due to the increased temperature and penetrates the fissures of the root canal.

Advantages:

- ▶ Facile on filing
- ▶ Removal of smear layer in canal
- ▶ Ideal product for cleansing and preparation of canal by using of files and rimers
- ▶ Appropriate for tight canals in posterior tooth
- ▶ Removal of inorganic materials by chelating
- ▶ Softening of dentin canal

Precautions:

- ▶ Do not use in similar cases.
- ▶ At the end of the work, the remainder of the material should be completely removed from the Patient mouth.
- ▶ Discard contact with skin and eyes.
- ▶ INGESTION: Wash out mouth with water.

- ▶ Do not induce Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Morva Prep
- ▶ Use eye protection, mask and gloves when using Morva Prep
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only

Warning:

- ▶ Do not use after expiration date
- ▶ Keep out of reach of children

Direction for Use:

- ▶ Isolated the treated tooth with rubber dam. Fill the tooth cavity with the product or place it directly on a canal tool.
- ▶ Mechanical working on the canal should be performed while rinsing it with sodium hypochlorite solution which increases the efficiency of the product.
- ▶ After finishing the treatment it is recommended to rinse the product well out of the canals and the tooth cavity using the sodium hypochlorite until the bubbling stops.
- ▶ Then wash with the serum.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance Store in a cool, dry place.
- ▶ Avoid exposure to direct light and heat.
- ▶ Store in original packaging at temperature between 5-25 °C.
- ▶ Once container is open, the expiry date doesn't change if it is securely closed after each use.
- ▶ Do not use Morva Prep cream for patients with hypersensitivity to the product's components.
- ▶ Do not leave Morva Prep cream in the canal or in the tooth cavity between the visits. It may soften the dentin too much.

					
Consult instructions for use	Indicates the batch code	Catalogue number	Indicates the date after which the medical device not to be used.	Indicates a medical device that needs protection from light sources.	Indicates a medical device that needs to be protected from moisture.
					
Do not use if Package is Damaged	Indicates the medical device manufacturer (Name & Address)	Indicates a medical device that has not been subjected to a sterilization process.	Indicates the device intended for use by professional person only.	Indicates that the medical device can cause discomfort and reddening, irritation, or blistering of the skin	Indicates the Temperature Limit

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